

Tapered Internal Drill Usage Chart

Description	Part Number	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	
2.0mm Starter Drill	TSD2020																					
2.5mm Depth Drill	TSD2025																					
2.0mm Starter Drill, Extended Shank	TSD4020																					
2.5mm Depth Drill, Extended Shank	TSD4025																					
2.5mm, 7.5mm Stop	TSD202507																					
2.5mm, 9mm Stop	TSD202509																					
2.5mm, 10.5mm Stop	TSD202510																					
2.5mm, 12mm Stop	TSD202512																					
2.5mm, 15mm Stop	TSD202515																					
3.2mm Width Increasing Drill	TSD2032																					
3.2mm Width Increasing Drill, Extended Shank	TSD4032																					
3.8mm Crestal Bone Drill	TSC2038																					
3.8mm Bone Tap	TST2038																					
3.7mm Width Increasing Drill	TSD2037																					
3.7mm Width Increasing Drill, Extended Shank	TSD4037																					
4.1mm Width Increasing Drill	TSD2041																					
4.1mm Width Increasing Drill, Extended Shank	TSD4041																					
4.6mm Crestal Bone Drill	TSC2046																					
4.6mm Bone Tap	TST2046																					
4.7mm Width Increasing Drill	TSD2047																					
4.7mm Width Increasing Drill, Extended Shank	TSD4047																					
5.4mm Width Increasing Drill	TSD2054																					
5.4mm Width Increasing Drill, Extended Shank	TSD4054																					
5.8mm Crestal Bone Drill	TSC2058																					
5.8mm Bone Tap	TST2058																					

Important Note: All drills should be replaced after they have been used 12 to 20 times, depending upon bone density.

Shop on-line at www.biohorizons.com